 MODE D'EMPLOI

 INSTRUCTION MANUAL

VAXPRO™

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2 ml / 5 ml

Veto Chirurgical

Chorfech 8, Sidi Thabet

2057 Ariana

Tunisia

Tel : +216 71 552 064

www.vetochirurgical.com

contact@vetochirurgical.com

VAXPRO™

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VETO CHIRURGICAL

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by Veto Chirurgica

VETO CHIRURGICAL

AUTOMATIC SELF-FILLING SYRINGE FOR INJECTION

This automatic vaccination syringe is designed for the repeated, precise and consistent administration of vaccines and injectable solutions for veterinary use.

Specifications


- Robust and durable metal construction
- Glass cylinder offering excellent visibility of the product
- Continuously adjustable dosage
- Two models available: 2 ml and 5 ml (in 0.2 ml increments)
- Compact and ergonomic design
- Metal feed connector with threaded system
- Seals ensuring a secure connection
- Aspiration needle integrated into the feed system
- Product container included
- Option to use a compatible bottle directly
- Luer Lock connector for secure attachment of the injection needle
- Protective ring for the Luer Lock connector

Scope of use

The syringe is intended for the administration of injectable products for veterinary use. The preparation of the product, the dosage, the mode of administration and the storage conditions must always comply with the recommendations of the manufacturer of the medicine or vaccine being used.

Main components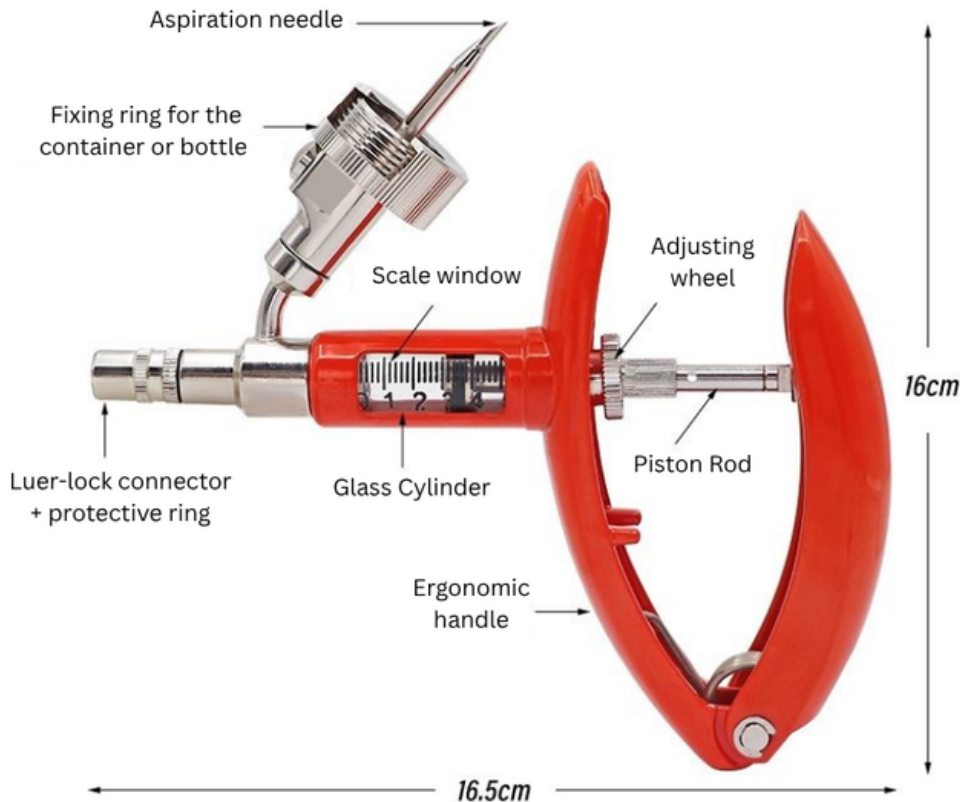

Aspiration needle

The aspiration needle is sharp and can cause injury.

Handle the connection system with care and avoid any contact with the needle.

Assembly and preparation

1) MOUNTING THE INJECTION NEEDLE

Before proceeding, check that the needle is suitable for the product and the intended route of administration.

Never use a damaged or loose needle

- Unscrew the protective ring from the connector.
- Access the Luer Lock connector.
- Screw the injection needle onto the Luer Lock connector.
- Tighten the needle until it is securely fastened.
- Screw the protective ring back onto the connector.

2) FITTING THE CONTAINER OR BOTTLE

The automatic syringe can also be used directly with a bottle that is compatible with the connection system.

Never force an incompatible bottle onto the syringe

- Remove the protective cap from the aspiration needle.
- Fill the container with the product to be administered.
- Before attaching the bottle, pierce its cap with a sterile needle.
- Position the container on the feed connector or the vaccine bottle.
- Screw the container or bottle onto the connector.
- Tighten the securing ring properly.
- Check that the container/bottle is secure and that there are no leaks.

3) ADJUSTING THE DOSAGE

- Turn the dosing wheel in the desired direction.

Dosing ranges

- 2 ml version: 0.2 ml increments
- 5 ml version: 0.2 ml increments

Cleaning and maintenance

1) CLEANING

*Cleaning must be carried out after each use.
Residues of vaccine, medication or injectable solution may dry out inside the circuit
and cause the system to malfunction or become blocked.*

- Remove the container or bottle and the injection needle.
- Rinse the system with clean water.
- Circulate sufficient liquid through the system to remove any product residue.
- Repeat the rinsing process until the system is clean.
- Drain off any excess water.
- Leave the syringe to dry completely.

2) LUBRIFICATION

- Once dry, apply a small amount of silicone oil to the parts requiring lubrication.
- Operate the mechanism several times to distribute the lubricant evenly.
- Regular lubrication helps to keep the mechanism running smoothly and extends the service life of the components.

3) INSPECTION AND MAINTENANCE

- Replace any part that shows signs of cracking, deformation, significant damage or a leak.

Veto Chirurgical

Chorfech 8, Sidi Thabet

2057 Ariana

Tunisia

Tel : +216 71 552 064

www.vetochirurgical.com

contact@vetochirurgical.com